

NEW TEST – Available Now

[Click for Pricing](#)

SSTR2 by Immunohistochemistry

3016782, SSTR2 IHC

Specimen Requirements:

Patient Preparation:

Collect: Tissue or cells.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a Tissue Transport Kit (ARUP supply #47808 highly recommended) available online through eSupply using ARUP Connect or contact ARUP Client Services at 800-522-2787 (Min: 2 slides). If sending precut slides, do not oven bake.

Transport Temperature:

Unacceptable Conditions: Tissue or cells not processed and placed in a paraffin block; serum, blood, or other body fluids; tissue not mounted on positively charged slides.

Remarks: IMMUNOHISTOCHEMISTRY ORDERING AND SUBMISSION DETAILS: Submit electronic request. If you do not have electronic ordering capability, use an ARUP Immunohistochemistry Stain Form (#32978) with an ARUP client number. For additional technical details, contact ARUP Client Services at 800-522-2787.

Stability: Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

Methodology: Immunohistochemistry

Performed: Mon-Fri

Reported: 1-3 days

Note: This test is performed as a stain and return (technical) service only.

CPT Codes: 88342

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Test Number	Components	Reference Interval
-------------	------------	--------------------

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.