

NEW TEST

[Click for Pricing](#)

Anti-Mullerian Hormone With Menopausal Status (MenoCheck)

3016862, AMH MENO

Specimen Requirements:

Patient Preparation:

Collect: Serum separator tube. Also acceptable: Plain red or green (lithium heparin).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum to an ARUP standard transport tube. (Min: 0.2 mL)

Transport Temperature: Frozen.

Unacceptable Conditions:

Remarks:

Stability: Ambient: Unacceptable; Refrigerated: 1 week; Frozen: 3 weeks (avoid repeated freeze/thaw cycles)

Methodology: Quantitative Enzyme-Linked Immunosorbent Assay

Performed: Sun-Sat

Reported: 1-3 days

Note:

CPT Codes: 82166

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

MenoCheck is an FDA-cleared test intended to be used as an aid in the determination of menopausal status in people between 42 to 62 years of age that were assigned female at birth. This test should be used in conjunction with other clinical and laboratory findings. This test should not be used to assess a menstruating individual's fertility status or for use in monitoring or predicting the ovarian response in menstruating individuals undergoing or planning to undergo fertility treatments.

Reference Interval:

Test Number	Components	Reference Interval		
	Anti-Mullerian Hormone, Menopause			
		Age	Reference Intervals (ng/mL)	
		40-45 years	Less than or equal to 6.282	
		46-50 years	Less than or equal to 0.064	
		Postmenopausal	Less than or equal to 0.003	

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.