

TEST CHANGE

Anti-Mullerian Hormone With Menopausal Status (MenoCheck)

3016862, AMH MENO

Specimen Requirements:

Patient Preparation:

Collect: Serum separator tube. Also acceptable: Plain red or green (lithium heparin).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum to an ARUP standard transport tube. (Min: 0.32 mL)

Transport Temperature: Frozen.

Unacceptable Conditions:

Remarks: Specimens must be from individuals 42-62 years of age.

Stability: Ambient: Unacceptable; Refrigerated: 1 week; Frozen: 3 weeks (avoid repeated freeze/thaw cycles)

Methodology: Quantitative Enzyme-Linked Immunosorbent Assay (ELISA)

Note:

CPT Codes: 82166

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

MenoCheck is an FDA-cleared test intended to be used as an aid in the determination of menopausal status in people between 42 to 62 years of age that were assigned female at birth. This test should be used in conjunction with other clinical and laboratory findings. This test should not be used to assess a menstruating individual's fertility status or for use in monitoring or predicting the ovarian response in menstruating individuals undergoing or planning to undergo fertility treatments.

Reference Interval:

Test Number	Components	Reference Interval	
	Anti-Mullerian Hormone, Menopause		
		Age	Reference Intervals (ng/mL)
		40-45 years	Less than or equal to 6.282
		46-50 years	Less than or equal to 0.064
		Postmenopausal	Less than or equal to 0.003

