

NEW TEST – Available Now[Click for Pricing](#)**Allergen, Food, Sesame Seed Component rSes i 1**

3017046, SESAME COM

Specimen Requirements:**Patient Preparation:** Multiple patient encounters should be avoided.**Collect:** Serum separator tube.**Specimen Preparation:** Separate serum from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum to an ARUP standard transport tube. (Min: 0.25 mL). For multiple allergen orders refer to "Allergen Specimen Collection Instructions" at www.aruplab.com/testing/resources/specimen.**Transport Temperature:** Refrigerated.**Unacceptable Conditions:** Hemolyzed, icteric, or lipemic specimens.**Remarks:****Stability:** After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year**Methodology:** Quantitative ImmunoCAP Fluorescent Enzyme Immunoassay**Performed:** Sun-Sat**Reported:** 1-3 days**Note:****CPT Codes:** 86008**New York DOH Approval Status:** This test is New York DOH approved.**Interpretive Data:**

Allergen results of 0.10-0.34 kU/L are intended for specialist use as the clinical relevance is undetermined. Even though increasing ranges are reflective of increasing concentrations of allergen-specific IgE, these concentrations may not correlate with the degree of clinical response or skin testing results when challenged with a specific allergen. The correlation of allergy laboratory results with clinical history and in vivo reactivity to specific allergens is essential. A negative test may not rule out clinical allergy or even anaphylaxis.

Reporting Range (reported in kU/L)	Probability of IgE Mediated Clinical Reaction	Class Scoring
Less than 0.10	No significant level detected	0
0.10 - 0.34	Clinical relevance undetermined	0/1
0.35 - 0.70	Low	1
0.71 - 3.50	Moderate	2
3.51 - 17.50	High	3
17.51 - 50.00	Very high	4
50.01 - 100.00	Very high	5
Greater than 100.00	Very high	6

Reference Interval:

Test Number	Components	Reference Interval
	Allergen, Food, Sesame rSes i 1 IgE	Less than or equal to 0.09 kU/L
	Allergen, Food, Sesame rSes i 1 IgE	

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.