

NEW TEST

[Click for Pricing](#)

PD-L1 22C3 by IHC
3017615, PDL1 22C3

Specimen Requirements:

Patient Preparation:

Collect: Tumor tissue

Specimen Preparation: Formalin fixed (10% neutral buffered formalin) and paraffin embedded (FFPE) specimen. Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a tissue transport kit (ARUP supply #47808 required), available online through eSupply using ARUP Connect(TM) or contact ARUP Client Services at 800-522-2787. (Min: 3 slides) If sending precut slides, do not oven bake.

Transport Temperature: Room temperature. Also acceptable: Refrigerated. Ship in cooled container during summer months.

Unacceptable Conditions: Paraffin block with no tumor tissue remaining; specimens fixed in any fixative other than 10% neutral buffered formalin. Decalcified specimens. Specimens with fewer than 100 viable tumor cells.

Remarks: Include surgical pathology report and indicate tissue site with the test order. For additional technical details, please contact ARUP Client Services at 800-522-2787. If multiple specimens (blocks or slides) are sent to ARUP, they must be accompanied by one of the following: an order comment indicating that the ARUP pathologist should choose the specimen most appropriate for testing (e.g., "Choose best block"), or individual orders for each sample submitted. A Pathologist Block Selection Fee (ARUP test code 3002076) will be added to orders that utilize the first option. If multiple specimens are sent to ARUP without a request for pathologist block/slide selection or individual orders, they will be held until clarification is provided.

Stability: Paraffin block: Ambient indefinitely, Refrigerated indefinitely, Frozen: Unacceptable Precut NSCLC slides: Ambient 6 months, Refrigerated 6 months, Frozen: Unacceptable Precut gastric/GEJ adenocarcinoma slides: Ambient: 5 months, Refrigerated 5 months, Frozen: Unacceptable Precut ESCC slides: Ambient 1 month, Refrigerated 4.5 months, Frozen: Unacceptable Precut cervical cancer slides: Ambient 1 month,

Refrigerated 5 months, Frozen: Unacceptable Precut HNSCC slides: Ambient 4 months, Refrigerated 6 months, Frozen: Unacceptable Precut TNBC slides: Ambient 4 months, Refrigerated 7.5 months, Frozen: Unacceptable

Methodology:	Immunohistochemistry (IHC)
Performed:	Mon-Fri
Reported:	3-7 days
Note:	This test code includes pathologist interpretation. At least 100 viable tumor cells are required for interpretation. Submission of slides that have been oven baked, or specimens that are decalcified and/or fixed in any fixative other than 10% neutral buffered formalin are not validated and should be interpreted with caution.
CPT Codes:	88360
New York DOH Approval Status:	Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible.
Interpretive Data:	Refer to report.
Reference Interval:	

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.