

TEST CHANGE

PD-L1 22C3 by IHC

3017615, PDL1 22C3

Specimen Requirements:

Patient Preparation:

Collect: Tumor tissue

Specimen Preparation: Formalin fixed (10% neutral buffered formalin) and paraffin embedded (FFPE) specimen. Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a tissue transport kit (ARUP supply #47808 required), available online through eSupply using ARUP Connect(TM) or contact ARUP Client Services at 800-522-2787. (Min: 3 slides) If sending precut slides, do not oven bake.

Transport Temperature: Room temperature. Also acceptable: Refrigerated. Ship in cooled container during summer months.

Unacceptable Conditions: Paraffin block with no tumor tissue remaining; specimens fixed in any fixative other than 10% neutral buffered formalin. Decalcified specimens. Specimens with fewer than 100 viable tumor cells.

Remarks: Include surgical pathology report and indicate tissue site with the test order. For additional technical details, please contact ARUP Client Services at 800-522-2787. If multiple specimens (blocks or slides) are sent to ARUP, they must be accompanied by one of the following: an order comment indicating that the ARUP pathologist should choose the specimen most appropriate for testing (e.g., "Choose best block"), or individual orders for each sample submitted. A Pathologist Block Selection Fee (ARUP test code 3002076) will be added to orders that utilize the first option. If multiple specimens are sent to ARUP without a request for pathologist block/slide selection or individual orders, they will be held until clarification is provided.

Stability: Paraffin block: Ambient indefinitely, Refrigerated indefinitely, Frozen: Unacceptable Precut NSCLC slides: Ambient 6 months, Refrigerated 6 months, Frozen: Unacceptable Precut gastric/GEJ adenocarcinoma slides: Ambient: 5 months, Refrigerated 5 months, Frozen: Unacceptable Precut ESCC slides: Ambient 1 month, Refrigerated 4.5 months, Frozen: Unacceptable Precut cervical cancer slides: Ambient 1 month, Refrigerated 5 months, Frozen: Unacceptable Precut HNSCC

slides: Ambient 4 months, Refrigerated 6 months, Frozen:
Unacceptable Precut TNBC slides: Ambient 4 months,
Refrigerated 7.5 months, Frozen: Unacceptable

Methodology:	Immunohistochemistry (IHC)
Performed:	Mon-Fri
Reported:	3-7 days
Note:	This test code includes pathologist interpretation. At least 100 viable tumor cells are required for interpretation. Submission of slides that have been oven baked, or specimens that are decalcified and/or fixed in any fixative other than 10% neutral buffered formalin are not validated and should be interpreted with caution.
CPT Codes:	88360
New York DOH Approval Status:	This test is New York DOH approved.
Interpretive Data:	Refer to report.
Reference Interval:	

HOTLINE NOTE: There is a reflexive pattern change associated with this test. One or more orderable or component has been added or removed to the reflexive pattern. Refer to the Hotline Test Mix for interface build information.