

NEW TEST

[Click for Pricing](#)

Lacosamide, Serum
3017887, LACOSA-S

Specimen Requirements:

Patient Preparation: Timing of specimen collection: Predose (trough) draw at steady-state concentration.

Collect: Plain red.

Specimen Preparation: Separate serum from cells within 2 hours of collection. Transfer 1 mL serum to an ARUP standard transport tube. (Min: 0.5 mL)

Transport Temperature: Refrigerated: Also acceptable: Room temperature or frozen.

Unacceptable Conditions: Whole blood. Gel separator tubes, light blue (citrate), or yellow (SPS or ACD solution).

Remarks:

Stability: Ambient: 48 hours; Refrigerated: 1 week; Frozen: 1 month 3 freeze/thaw cycles

Methodology: Quantitative Enzyme Immunoassay (EIA)

Performed: Sun-Sat

Reported: 1-2 days

Note:

CPT Codes: 80235

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Test Number	Components	Reference Interval

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.