

NEW TEST

[Click for Pricing](#)

Vancomycin Level, Trough, Serum

3018758, VANCOTR

Specimen Requirements:

Patient Preparation: Collect specimen 30 minutes before next dose.

Collect: Plain red

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum to an ARUP standard transport tube. (Min: 0.25 mL) Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: Refrigerated. Also Acceptable: Frozen

Unacceptable Conditions:

Remarks:

Stability: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year

Methodology: Quantitative Kinetic Interaction of Microparticles in Solution (KIMS)

Performed: Varies

Reported: 3-5 days

Note:

CPT Codes: 80202

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Test Number	Components	Reference Interval

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.