

TEST CHANGE

Extended Myositis Panel

3018867, MYOS EXT2

Specimen Requirements:

Patient Preparation:

Collect: Serum separator tube (SST), red top tube

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 4 mL to ARUP standard transport tube. (Min: 3 mL)

Transport Temperature: Refrigerated

Unacceptable Conditions: Hemolyzed, hyperlipemic, icteric, heat-treated, or contaminated specimens.

Remarks:

Stability: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 month

Methodology: ~~Semi-Quantitative Enzyme-Linked Immunosorbent Assay (ELISA)~~ / Qualitative Immunoprecipitation / Semi-Quantitative Multiplex Bead Assay / Qualitative Immunoblot / Semi-Quantitative Indirect Fluorescent Antibody (IFA) / Qualitative Particle-Based Multianalyte Technology (PMAT)

Note: Antibodies: Mi-2, PL-7, PL12, P155/140, EJ, Ku, OJ, PM/ScI, SRP, Smith/RNP, Ro52, Ro60, Jo-1, U3 Fib, SAE1, NXP2, MDA5, TIF1-gamma, ANA, Ha, Ks, Zo

If HMGCR antibody, IgG is positive, additional testing to follow.

CPT Codes: 83516 x13; 84182 x3; 86039; 86235 x6; if reflexed, add 82397

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Refer to report.

Component	Interpretation
SSA-52 (Ro52) (ENA) Antibody, IgG	29 AU/mL or less: Negative 30-40 AU/mL: Equivocal 41 AU/mL or greater: Positive
SSA-60 (Ro60) (ENA) Antibody, IgG	29 AU/mL or less: Negative 30-40 AU/mL: Equivocal 41 AU/mL or greater: Positive
Smith/RNP (ENA) Antibody, IgG	29 AU/mL 19 Units or less: Negative 30-20-39 Units: Weak Positive 40 AU/mL: Equivocal 41 AU/mL-80 Units: Moderate

Jo-1 Antibody, IgG	<p>Positive 81 Units or greater: Strong Positive</p> <p>29 AU/mL or less: Negative 30-40 AU/mL: Equivocal 41 AU/mL or greater: Positive</p>
--------------------	--

Reference Interval:

Test Number	Components	Reference Interval
	Antinuclear Antibody (ANA), HEp-2, IgG	Less than 1:80
	EJ (glycyl-tRNA synthetase) Antibody	Negative
	Fibrillarin (U3 RNP) Ab, IgG	Negative
	Ha (tyrosyl-tRNA synthetase) Ab	Negative
	HMGCR Antibody Screen	Negative
	Jo-1 (Histidyl-tRNA Synthetase) Ab, IgG	40 AU/mL or less
	Ks (asparaginyl-tRNA synthetase) Ab	Negative
	Ku Antibody	Negative
	MDA5 (CADM-140) Ab	Negative
	Mi-2 (nuclear helicase protein) Antibody	Negative
	NXP2 (Nuclear matrix protein-2) Ab	Negative
	OJ (isoleucyl-tRNA synthetase) Antibody	Negative
	P155/140 Antibody	Negative
	PL-12 (alanyl-tRNA synthetase) Antibody	Negative
	PL-7 (threonyl-tRNA synthetase) Antibody	Negative
	PM/Scl 100 Antibody, IgG	Negative
	SAE1 (SUMO activating enzyme) Ab	Negative
	Smith/RNP (ENA) Ab, IgG	40 AU/mL 19 Units or less
	SRP (Signal Recognition Particle) Ab	Negative
	SSA-52 (Ro52) (ENA) Antibody, IgG	40 AU/mL or less
	SSA-60 (Ro60) (ENA) Antibody, IgG	40 AU/mL or less
	TIF-1 gamma (155 kDa) Ab	Negative
	Zo (phenylalanyl-tRNA synthetase) Ab	Negative