

NEW TEST

Click for Pricing

Autoimmune Movement Disorder Panel, CSF

3018966, AIMDC 2	
Specimen Requirements:	
Patient Preparation:	
Collect:	CSF
Specimen Preparation:	Transfer three 1 mL CSF aliquots to ARUP standard transport tubes. (Min: 0.5 mL/aliquot)
Transport Temperature:	Frozen
Unacceptable Conditions:	Fluid other than CSF. Grossly hemolyzed specimens
Remarks:	
Stability:	After separation from cells: Ambient: 24 hours; Refrigerated: 1 week; Frozen: 1 month (avoid repeated freeze/thaw cycles)
Methodology:	Semi-Quantitative Cell-Based Indirect Fluorescent Antibody / Semi-Quantitative Indirect Fluorescent Antibody (IFA) / Qualitative Immunoblot / Semi-Quantitative Enzyme-Linked Immunosorbent Assay (ELISA)
Performed:	Varies
Reported:	3-10 days
Note:	If NMDA CSF antibody IgG is positive, then titer will be performed. Additional charges apply. If CV2 CSF antibody IgG is positive, then titer will be added. Additional charges apply. PCCA/ANNA CSF antibody IgG is screened by IFA. If the IFA screen is indeterminate, then a Neuronal Nuclear Antibodies (Hu, Ri, Yo, and Tr/DNER) IgG by Immunoblot will be performed. If the IFA screen is positive at 1:10 or greater, then a PCCA/ANNA antibodies titer and Neuronal Nuclear Antibodies (Hu, Ri, Yo, Tr/DNER) IgG by Immunoblot will be performed. Additional charges apply. If PCCA is detected ITPR1 antibody IgG will be added and if positive, then titer will be added. Additional charges apply. If LG11 CSF antibody IgG is positive, then titer will be added. Additional charges apply. If CASPR2 CSF antibody IgG is positive, then titer will be added. Additional charges apply. If DPPX CSF antibody IgG by IFA is positive, then titer will be added. Additional charges apply. If AMPA CSF antibody IgG is positive, then titer will be added. Additional



	charges apply. If GABA-BR CSF antibody IgG is positive, then titer will be added. Additional charges apply. If GABA-AR CSF antibody IgG by IFA is positive, then titer will be added. Additional charges apply. If mGluR1 CSF antibody IgG by IFA is positive, then titer will be added. Additional charges apply. If IgLON5 CSF antibody IgG by IFA is positive, then titer will be added. Additional charges apply. If KLHL11 antibody IgG by IFA is positive, then titer will be added. Additional charges apply.
CPT Codes:	86341; 84182 x3; 86255 x12; if reflexed add 84182 x4; 86255; 86256 per titer
New York DOH Approval Status:	This test is New York DOH approved.
Interpretive Data:	
Reference Interval:	
Test Components Number	Reference Interval

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.