

NEW TEST

[Click for Pricing](#)

High-Grade B-Cell Lymphoma Reflex Panel by FISH, Tissue

3019135, HGBCL RFLX

Specimen Requirements:

Patient Preparation:

Collect: Tumor tissue.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin-embed specimen. Protect paraffin block from excessive heat. Transport tissue block or 8 unstained (3-micron thick sections) positively charged slides in a tissue transport kit (ARUP supply #47808) available online through eSupply using ARUP Connect™ or contact ARUP Client Services at 800-522-2787 (kit is recommended but not necessary). (Min: 4 slides)

Transport Temperature: Room temperature or refrigerated. Ship in cooled container during summer months.

Unacceptable Conditions: Specimens fixed or processed in alternative fixatives (alcohol, Prefer) or heavy metal fixatives (B-4 or B-5). No tumor in tissue. Decalcified specimens.

Remarks: Include surgical pathology report. If multiple specimens (blocks or slides) are sent to ARUP, they must be accompanied by one of the following: an order comment indicating that the ARUP pathologist should choose the specimen most appropriate for testing (e.g., "Choose best block"), or individual orders for each sample submitted. A Pathologist Block Selection Fee (ARUP test code 3002076) will be added to orders that utilize the first option. If multiple specimens are sent to ARUP without a request for pathologist block/slide selection or individual orders, they will be held until clarification is provided.

Stability: Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

Methodology: Fluorescence in situ Hybridization (FISH)

Performed: Mon-Fri

Reported: 3-7 days

Note: If Aggressive B-Cell Lymphoma Reflex Panel by FISH is

positive, then IGH-BCL2 Fusion, t(14;18) by FISH (ARUP test code 3001298) and BCL6 (3q27) Gene Rearrangement by FISH (ARUP test code 3001311) will be added. If Aggressive B-Cell Lymphoma Reflex Panel by FISH is negative, then 11q Aberrations by FISH (ARUP test code 3019126) will be added. Additional charges apply.

CPT Codes: 88366; if reflexed, add 88366; 88377

New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible.

Interpretive Data:

Refer to report.

Reference Interval:

By report

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.