

NEW TEST – Available Now

[Click for Pricing](#)

ICOS by Immunohistochemistry

3019353, ICOS_IHC

Specimen Requirements:

Patient Preparation:

Collect: Tissue or cells.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a tissue transport kit (ARUP supply #47808). Available online through eSupply using ARUP Connect(TM) or contact ARUP Client Services at 800-522-2787. (Min: 2 slides). If sending precut slides, do not oven bake.

Transport Temperature: Room temperature or refrigerated. Ship in cooled container during summer months.

Unacceptable Conditions: Specimens submitted with nonrepresentative tissue type. Depleted specimens.

Remarks: IMMUNOHISTOCHEMISTRY ORDERING AND SUBMISSION DETAILS : Submit electronic request. If you do not have electronic ordering capability, use an ARUP Immunohistochemistry Stain Form (#32978) with an ARUP client number. For additional technical details, contact ARUP Client Services at 800-522-2787.

Stability: Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

Methodology: Immunohistochemistry (IHC)

Performed: Mon-Fri

Reported: 1-3 days

Note: This test is performed as a stain and return (technical) service only.

CPT Codes: 88342

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Test Number	Components	Reference Interval
-------------	------------	--------------------

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.