

NEW TEST

[Click for Pricing](#)

Vitamin B7 (Biotin), Serum or Plasma

3020435, VITAMB7 SP

Specimen Requirements:

Patient Preparation:

Collect: Plain red, lavender (K2EDTA), or pink (K2EDTA)

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 2 mL serum or plasma to an ARUP standard transport tube. (Min: 0.7 mL)
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: Frozen

Unacceptable Conditions: Polymer gel separation tube (SST or PST).

Remarks:

Stability: Ambient: Unacceptable; Refrigerated: 24 hours; Frozen: 2 weeks

Methodology: Quantitative High Performance Liquid Chromatography-Tandem Mass Spectrometry

Note:

CPT Codes: 84591

New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible.

Interpretive Data:

Reference Interval:

Refer to report

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.