

NEW TEST

[Click for Pricing](#)

Drug Panel, Screen With Reflex to Quantitation, Urine

3020808, DSCR U

Specimen Requirements:

Patient Preparation:

Collect: Random urine.

Specimen Preparation: Transfer 8 mL urine with no additives or preservatives in ARUP standard transport tubes. (Min: 4 mL)

Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens exposed to repeated freeze/thaw cycles.

Remarks:

Stability: Ambient: 1 week; Refrigerated: 1 month; Frozen: 1 month

Methodology: Qualitative Enzyme Multiplied Immunoassay Technique (EMIT) / Quantitative Gas Chromatography-Flame Ionization Detection / Quantitative Gas Chromatography-Mass Spectrometry (GC-MS) / Quantitative Liquid Chromatography-Tandem Mass Spectrometry

Note:

If the specimen screens positive, then confirmation/quantitation by GC/MS, GC/FID, and/or LC-MS/MS will be added to confirm result. Additional charges apply.

To order testing for individual opioids, refer to Fentanyl and Metabolite, Urine, Quantitative (ARUP test code 0092570), Buprenorphine and Metabolites, Urine, Quantitative (ARUP test code 2010092), Meperidine and Metabolite Quantitative, Urine (ARUP test code 3000248), Tramadol and Metabolite, Urine, Quantitative (ARUP test code 2002736). For the comprehensive panel, refer to Drug Profile, Targeted by Tandem Mass Spectrometry and Enzyme Immunoassay, Urine (ARUP test code 2007479).

CPT Codes: 80307; if reflexed, add 80321; 80325; 80345; 80346; 80349; 80353; 80354; 80358; 80359; 80361; 80365; 83992 (Reflexed Alt Code: G0480)

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Flagging indicates a positive result; not that the result is abnormal.

Presumptive negative results for drug(s) and/or drug metabolite(s) may indicate

noncompliance, inappropriate timing of specimen collection relative to drug administration, poor drug absorption, diluted/adulterated urine, or limitations of testing. The concentration at which the screening test can detect a drug or metabolite varies within a drug class.

For medical purposes only; not valid for forensic use.

Reference Interval:

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.