

NEW TEST

[Click for Pricing](#)

Drug Panel, Expanded Screen With Reflex to Quantitation, Urine

3020812, DSCR UEXP

Specimen Requirements:

Patient Preparation:

Collect: Random urine.

Specimen Preparation: Transfer 4 mL each into two (2) ARUP standard transport tubes of urine with no additives or preservatives. (Min: 2 mL each)

Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens exposed to multiple freeze/thaw cycles, Pharmaceutical preparation.

Remarks:

Stability: Ambient: 1 week; Refrigerated: 1 month; frozen: 1 month

Methodology: Qualitative Enzyme Multiplied Immunoassay Technique (EMIT) / Quantitative Gas Chromatography-Mass Spectrometry (GC-MS) / Quantitative Gas Chromatography-Mass Spectrometry (GC-MS) / Quantitative Liquid Chromatography-Tandem Mass Spectrometry

Note:

If the specimen screens positive, then confirmation/quantitation by GC/MS and/or LC-MS/MS will be added to confirm result. Additional charges apply.

CPT Codes: 80307; if reflexed, add 80321; 80325; 80345; 80346; 80348; 80349; 80353; 80354; 80358; 80359; 80361; 80365; 80368; 80369; 80372; 80373; 83992 (Reflexed Alt Code: G0480)

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Flagging indicates a positive result; not that the result is abnormal.

Presumptive negative results for drug(s) and/or drug metabolite(s) may indicate noncompliance, inappropriate timing of specimen collection relative to drug administration, poor drug absorption, diluted/adulterated urine, or limitations of testing. The concentration at which the screening test can detect a drug or metabolite varies within a drug class.

For medical purposes only; not valid for forensic use.

Drugs covered and range of cutoff concentrations	Screen
Amphetamines	Methodology: Immunoassay Positive Cutoff: 300 ng/mL
Barbiturates	Methodology: Immunoassay Positive Cutoff: 200 ng/mL
Benzodiazepines	Methodology: Immunoassay Positive Cutoff: 200 ng/mL
Buprenorphine	Methodology: Immunoassay Positive Cutoff: 5 ng/mL
Carisoprodol	Methodology: Immunoassay Positive Cutoff: 100 ng/mL
Cocaine	Methodology: Immunoassay Positive Cutoff: 150 ng/mL
Ethyl Glucuronide	Methodology: Immunoassay Positive Cutoff: 500 ng/mL
Fentanyl	Methodology: Immunoassay Positive Cutoff: 1 ng/mL
Opiates	Methodology: Immunoassay Positive Cutoff: 300 ng/mL
Oxycodone	Methodology: Immunoassay Positive Cutoff: 100 ng/mL
Methadone	Methodology:

Metabolite (EDDP)	Immunoassay Positive Cutoff: 100 ng/mL
Phencyclidine	Methodology: Immunoassay Positive Cutoff: 25 ng/mL
Tapentadol	Methodology: Immunoassay Positive Cutoff: 200 ng/mL
Tramadol	Methodology: Immunoassay Positive Cutoff: 100 ng/mL
THC (Cannabinoids)	Methodology: Immunoassay Positive Cutoff: 50 ng/mL
Zolpidem	Methodology: Immunoassay Positive Cutoff: 20 ng/mL

Reference Interval:

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.