

NEW TEST

[Click for Pricing](#)

Methadone Metabolite, Urine, Screen With Reflex to Quantitation

3020813, EDDP RFX U

Specimen Requirements:

Patient Preparation:

Collect: Random Urine

Specimen Preparation: Transfer 4 mL urine with no additives or preservatives to an ARUP standard transport tube. (Min: 2 mL)

Transport Temperature: Room temperature.

Unacceptable Conditions: Breast milk. Specimens exposed to repeated freeze/thaw cycles.

Remarks:

Stability: Ambient: 1 week; Refrigerated: 1 month; Frozen: 1 month

Methodology: Qualitative Enzyme Immunoassay (EIA) / Quantitative Liquid Chromatography-Tandem Mass Spectrometry

Note:

If the specimen screens positive, then Methadone and Metabolite, Urine, Quantitative (ARUP test code 0090362) will be added to confirm result. Additional charges apply.

Compare to Pain Management, Methadone, Quantitative, with medMatch, Urine; Pain Management, Methadone, with Confirmation with medMatch, Urine.

CPT Codes: 80307; if reflexed, add 80358 (Reflexed Alt Code: G0480)

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Methodology: Immunoassay

Positive Cutoff: 100 ng/mL

Reference Interval:

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.