

NEW TEST

[Click for Pricing](#)

High Molecular Weight Kininogen (HMWK), Plasma

3020924, HMW KININ

Specimen Requirements:

Patient Preparation: Do not draw from an arm with a heparin lock or heparinized catheter.

Collect: Light blue (sodium citrate).

Specimen Preparation: Separate from cells within 4 hours of collection. Transfer 1 mL plasma to an ARUP standard transport tube and freeze immediately. (Min: 1 mL)
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: Frozen.

Unacceptable Conditions:

Remarks:

Stability: Ambient: 4 hours; Refrigerated: Unacceptable; Frozen: 6 months

Methodology: Clotting

Note:

CPT Codes: 85293

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Refer to report

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.