

## NEW TEST – Available Now

[Click for Pricing](#)

### LMO2 by Immunohistochemistry

3020992, LMO2\_IHC

#### Specimen Requirements:

##### Patient Preparation:

**Collect:** Tissue or cells.

**Specimen Preparation:** Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a tissue transport kit (ARUP supply #47808). Available online through eSupply using ARUP Connect(TM) or contact ARUP Client Services at 800-522-2787. (Min: 2 slides). If sending precut slides, do not oven bake.

**Transport Temperature:** Room temperature or refrigerated. Ship in cooled container during summer months.

**Unacceptable Conditions:** Specimens submitted with nonrepresentative tissue type. Depleted specimens.

**Remarks:** IMMUNOHISTOCHEMISTRY ORDERING AND SUBMISSION DETAILS : Submit electronic request. If you do not have electronic ordering capability, use an ARUP Immunohistochemistry Stain Form (#32978) with an ARUP client number. For additional technical details, contact ARUP Client Services at 800-522-2787.

**Stability:** Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

**Methodology:** Qualitative Immunohistochemistry (IHC)

**Note:** This test is performed as a stain and return only.

**CPT Codes:** 88342

**New York DOH Approval Status:** This test is New York DOH approved.

#### Interpretive Data:

#### Reference Interval:

**HOTLINE NOTE:** Refer to the Hotline Test Mix for interface build information.

