

NEW TEST

[Click for Pricing](#)

Allergen, Food, Hazelnut Component Cor a 14, IgE

3021006, COR A 14

Specimen Requirements:

Patient Preparation:	Multiple patient encounters should be avoided.
Collect:	Serum separator tube or plain red.
Specimen Preparation:	Separate serum from cells ASAP or within 2 hours of collection. Transfer 1.0 mL serum to an ARUP standard transport tube. (Min: 0.25 mL). For multiple allergen orders refer to "Allergen Specimen Collection Instructions" at www.aruplab.com/testing/resources/specimen .
Transport Temperature:	Refrigerated.
Unacceptable Conditions:	Hemolyzed, icteric, or lipemic specimens.
Remarks:	
Stability:	After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 month

Methodology: Quantitative ImmunoCAP Fluorescent Enzyme Immunoassay

Note:

CPT Codes: 86008

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Presence of allergen-specific IgE indicates sensitization, but it may not differentiate between tolerant and allergic individuals. Serology results alone cannot confirm or establish an allergy diagnosis. A negative test does not rule out clinical allergy or even anaphylaxis.

Reference Interval:

Test Number	Components	Reference Interval
	Allergen, Hazelnut Component, Cor a14, IgE	Less than or equal to 0.09 kU/L

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.