

NEW TEST

[Click for Pricing](#)

Allergen, Food, Soybean with Reflex to Components, IgE

3021008, SOYBEAN R

Specimen Requirements:

Patient Preparation:	Multiple patient encounters should be avoided.
Collect:	Serum separator tube or plain red.
Specimen Preparation:	Separate serum from cells ASAP or within 2 hours of collection. Transfer 2.0 mL serum to an ARUP standard transport tube. (Min: 1.0 mL). For multiple allergen orders refer to "Allergen Specimen Collection Instructions" at www.aruplab.com/testing/resources/specimen .
Transport Temperature:	Refrigerated.
Unacceptable Conditions:	Hemolyzed, icteric, or lipemic specimens.
Remarks:	
Stability:	After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 month
Methodology:	Quantitative ImmunoCAP Fluorescent Enzyme Immunoassay

Note:

This assay will initially test soybean whole allergen. If the whole allergen result is greater than or equal to 0.10 kU/L, Allergen, Food, Soybean Components IgE (Gly m 4, Gly m 5, and GLY m6) will be ordered. Additional charges apply.

CPT Codes: 86003; if reflexed, add 86008 x3

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Test Number	Components	Reference Interval
	Allergen, Food, Soybean IgE	Less than or equal to 0.34 kU/L

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.