

NEW TEST

[Click for Pricing](#)

Neurofilament Light Chain, Serum

3021400, NFLC S

Specimen Requirements:

Patient Preparation: Discontinue biotin supplements (vitamin B7 or B8, vitamin H, or coenzyme R) for 72 hours prior to collection.

Collect: Plain red or serum separator tube (SST).

Specimen Preparation: Transfer 1 mL serum to an ARUP standard transport tube. (Min: 0.7 mL)
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: Room temperature. Also acceptable: Refrigerated or frozen.

Unacceptable Conditions:

Remarks:

Stability: Ambient: 2 weeks; Refrigerated: 2 weeks; Frozen: 2 weeks

Methodology: Quantitative Electrochemiluminescent Immunoassay (ECLIA)

Note: Direct comparisons of absolute values can only be done on the same source fluid (serum or plasma).

CPT Codes: 83884

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Refer to report

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.